

Now in the clinical lab

BD Veritor™ System analyzes...



...All that
you see



and all that
you can't.

BD Veritor™ System

For Rapid Detection of RSV*



Helping all people
live healthy lives

- **Technology-verified Performance**— Incorporates state of the art technologies that deliver results you can trust
- **Multiple Sample Types**— Validated to be used with nasopharyngeal washes/aspirates, and nasopharyngeal swabs in transport
- **Convenient Kit Configuration**— Fully customized kit to test and deliver results in minutes
- **Objective Results**— Easy to read digital display eliminates subjective interpretation

*Respiratory Syncytial Virus

When RSV is suspected...

Detect. Analyze. Verify.

BD Veritor™ System – Proven Performance

Clinical Performance— BD Veritor RSV to Viral Cell Culture NPS				Clinical Performance— BD Veritor RSV to Viral Cell Culture NPWA			
	Viral Cell Culture				Viral Cell Culture		
BD Veritor RSV	P	N		BD Veritor RSV	P	N	
P	153	9*	162	P	152	15**	167
N	20	524	544	N	14	259	273
	173	533	706		166	274	440
Sensitivity: 153/173 = 88.4% Specificity: 524/533 = 98.3%				Sensitivity: 152/166 = 91.6% Specificity: 259/274 = 94.5%			

*Of the 9 BD Veritor RSV Positive, Viral Cell Culture negative specimens, 6 were positive by FDA cleared Prodesse® ProFlu™+ molecular assay (Data from package insert)

**Of the 15 BD Veritor RSV Positive, Viral Cell Culture negative specimens, 8 were positive by FDA cleared Prodesse® ProFlu™+ molecular assay (Data from package insert)

State of the Art Technologies

- Advanced Particle Technology enhances sensitivity, helping to improve test performance
- Adaptive Read Technology helps improve specificity to reduce false positive results

Convenient Workflow Delivers Results in Minutes



Easy sample processing
Prefilled unitized tube facilitates workflow



Ready in minutes
Test device is ready to insert into reader 10 minutes after sample is added



Insert and read
Simple one-touch button readies the reader for test device insertion



Objective results
Once the test device is inserted in the reader, an objective, digital test result is displayed in 10 seconds

Ordering Information			
Description	Cat. No.	Qty.	Description
BD Veritor™ System Reader	256055	1	BD Veritor™ System RSV Kit
			256042 30



BD Diagnostics
7 Loveton Circle
Sparks, MD 21152-0999
www.bd.com/ds

BD, BD Logo and BD Veritor are trademarks of Becton, Dickinson and Company
©2012 BD 223560 September 2012
Printed in USA

Prodesse and ProFlu are trademarks of Hologic Gen-Probe Incorporated

BD Veritor™ System

Changing the Way You View Rapid Testing

www.bd.com/ds/veritorsystem