



GIARDIA/CRYPTOSPORIDIUM QUIK CHEK

NEW Enteric Offering

Giardia and *Cryptosporidium* are the two most common waterborne intestinal parasites in the United States. The new *GIARDIA/CRYPTOSPORIDIUM QUIK CHEK* test improves upon traditional testing methods by eliminating complicated, labor-intensive processes all while maintaining high levels of accuracy. This rapid immunoassay simultaneously detects and differentiates *Giardia* cyst antigen and *Cryptosporidium* oocyst antigen, reducing the time and cost when compared to traditional methods.

One Test, Two Results

- Simultaneous detection and differentiation of
- *Giardia* and *Cryptosporidium* using one test device
- Less labor intensive than traditional methods
- Reimbursable for both organisms

Simple and Accurate Performance

- Results in 30 minutes without instrumentation
- Cost-effective
- No subjective reading of results
 - *Giardia*: Sensitivity 98.9% / Specificity 100%
 - *Crypto.*: Sensitivity 100% / Specificity 99.8%

GIARDIA/CRYPTOSPORIDIUM QUIK CHEK





Actual Size


To learn more about cost-effective, sensitive, rapid detection of *Giardia* and *Cryptosporidium*, contact your local Account Executive.


Easy to use

- 1** Add:
 -750 μ L diluent
 -1 drop conjugate
 - 25 μ L fresh sample

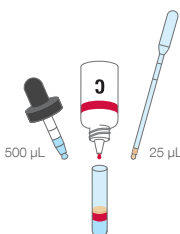
- 2** Mix & add 500 μ L to cassette, incubate 15 minutes

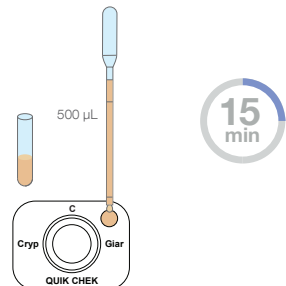

- 3** Add 300 μ L wash buffer

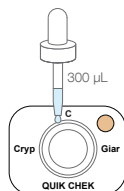
- 4** Add 2 drops substrate, read at 10 minutes after addition of substrate*

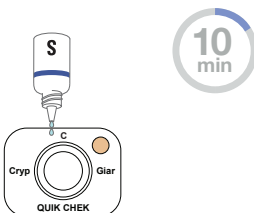


*A positive result may be interpreted at any time during the 10-minute incubation. A test cannot be ruled as negative or invalid until after the 10-minute incubation. Sample instructions are for fresh samples only. See package insert for full procedure instruction and preserved sample volumes.


Easy to read

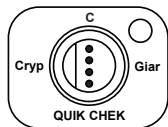


POSITIVE
Crypto. Result

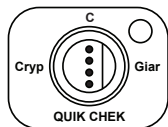

POSITIVE
Giardia Result

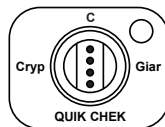

POSITIVE Crypto. &
POSITIVE Giardia Result

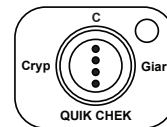

NEGATIVE Crypto. &
NEGATIVE Giardia Result

Order Information:

GIARDIA/CRYPTOSPORIDIUM QUIK CHEK

Catalog No. **23046789**

CPT Code: *Giardia* 87329 / *Cryptosporidium*

87328 CLIA: Moderate Complexity

Developed and
 Manufactured by:

TECHLAB

Alere cannot guarantee or promise coverage or payment for any particular item or service from any payer or health benefit plan. To be eligible for coverage, an item or service must be medically necessary for the individual patient, have been performed as reported, and appropriate documentation should be available in the patient's medical record. It is the individual provider's responsibility to determine appropriate, medically necessary coding, charges and claims for a particular service. Laws, regulations and payer policies regarding appropriate coding and payment levels can vary greatly from payer to payer and change over time. Alere recommends that providers contact their own regional payers to determine appropriate coding and charge or payment levels. Alere specifically disclaims liability or responsibility for the results or consequences of any actions taken in reliance on information present here.

Made in the U.S.A.

© 2014 Alere. All rights reserved. The Alere Logo and Alere are trademarks of the Alere group of companies. TECHLAB is a trademark of TECHLAB, Inc. under license. The Fisher Healthcare Logo is a trademark of Fisher Scientific Company, L.L.C. 3000234-01 06/14 BN0630148

In the United States:

For customer service, call 1-800-640-0640

To fax an order, use 1-800-290-0290

To order online: www.fisherhealthcare.com



A Thermo Fisher Scientific Brand