



Overcome the challenges of fecal white blood cell microscopy with the LEUKO EZ VUE™ immunochromatographic test.

Detect elevated levels of fecal lactoferrin, a marker of fecal leukocytes and an indicator of intestinal inflammation. With rapid results in only 10 minutes, patients can quickly be given the appropriate treatment and put on the path to better health.

- Minimal sample preparation
- Lateral flow assay format
- Easy-to-use
- Clear results in 10 minutes
- >90% correlation*




Actual size

To learn more about the **LEUKO EZ VUE™** test, contact your local Account Executive.

Easy to Use


- 1** Add 50 µl of specimen to 2.5 mL diluent

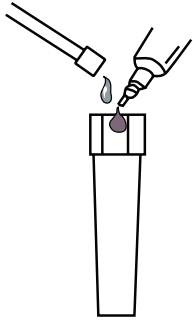

- 2** Add filter tip to test tube and mix (vortex)



- 3** Add 5 drops to sample well

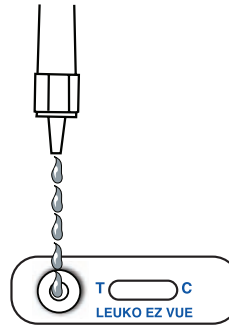

- 4** Read at 10 minutes



For complete instructions for use, see the package insert.

Easy to Read



Positive

One line on the Control (C) and one line on the Test (T).



Negative

One line on the Control (C) and **no** line on the Test (T).



Invalid

No lines or a single line on the Test (T) side.

Developed and
Manufactured by:



Order Information

LEUKO EZ VUE™

Catalog No: **23046530** – 25 Tests

CPT Code: 83630

*Refer to product insert for full details.

Alere cannot guarantee or promise coverage or payment for any particular item or service from any payer or health benefit plan. It is the individual provider's responsibility to determine appropriate, medically necessary coding, charges and claims for a particular service. Alere recommends that providers contact their own regional payers to determine appropriate coding and charge or payment levels. Alere specifically disclaims liability or responsibility for the results or consequences of any actions taken in reliance on information present here.

© 2014 Alere. All rights reserved. The Alere Logo and Alere are trademarks of the Alere group of companies. *LEUKO EZ VUE* and *TECHLAB* are trademarks of *TECHLAB, Inc.*, under license. The Fisher Healthcare Logo is a trademark of Fisher Scientific Company, L.L.C. 3000235-01 06/15 BN0630149

In the United States:

For customer service, call 1-800-640-0640

To fax an order, use 1-800-290-0290

To order online: www.fisherhealthcare.com



A Thermo Fisher Scientific Brand